Gebruikershandleiding
User Manual
Gebrauchsanleitung
Brugervejledning

Disposable Tilbanden

Patient specific Slings

Einweggurte

Løftesejl-Èn-Patient

Inhoudsopgave - Table of content

Inhaltverzeichnis - Indholdsfortegnelse

Nederlands	4
English	8
Deutsch	12
Dansk.....	16

Bedankt

Geachte klant,

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Thank you

Dear customer,

Thank you for choosing Tailored Perfection™ patient specific slings from MSP Medical Products B.V.

This sling is part of a wider catalogue of slings which we offer under our Tailored Perfection™ label and who'll aid you in carrying out safe and comfortable transfers.

Please read this manual carefully before first use and store properly.

More information can be found on www.mspmedicalproducts.com

NL

EN

DE

DK

Important!

These instructions are for guidance only and are not intended to be exhaustive. When in doubt, always consult your manual handling trainer or manager. All moving and handling tasks of the caretaker are to be carried out in accordance with approved patient handling techniques and after a full assessment by a certified person, taking into account the state of the caretaker, the task, the equipment and the environment in which the operation will take place. The caregiver must be fully practiced and competent in the use of the medical equipment that will be used, before using these slings with a patient.

NEVER PRATICE WITH A PATIENT

Inspections before each use of the sling

Check the sling on the following points before each use:

- Signs of wear and tear
- Check all stitching for any signs of wear
- Check in particular all orange stitching around the loops and make sure they are present and there is no sign of damage
- Check for presence of the green rectangular label. This label will disappear when the sling is washed. If so DO NOT USE the sling.
- Inspect the patientlift according to the manufacturer's instructions before use

DO NOT use the sling if one or more of the above points is noticed.

DO NOT use the sling if damaged.

DO NOT use the sling if the green rectangular label is missing.

Size	Colour Loops	Weight Indication	Length Indication	Article Number
MS Medium/Small	Red	45-70kg	130-145cm	MSP-TB-DP-00130HS
M Medium	Yellow	60-90kg	145-165cm	MSP-TB-DP-00140HS
L Large	Green	80-130kg	165-185cm	MSP-TB-DP-00150HS
XL Extra Large	Blue	110-200kg	180-215cm	MSP-TB-DP-00170HS

Instructions before use

When the sling is first used, write the patients name in the box provided on the information label and the date of first use in the adjacent box.

When reusing a sling, check the patients name corresponds to the one on the sling.

When lifting from a sitting position, please ensure that:

1. The sling is centrally positioned underneath the patient.
2. The leg straps are positioned under each thigh, ensuring there is no creasing or rucking.

Alternatively, when lifting from a lying position, please ensure that:

1. After rolling the patient onto the sling, the top of the commode aperture is level with the base of the spine.
2. The leg straps are positioned under each thigh, ensuring there is no creasing or rucking

Attach the loops of the leg straps to the spreader bar crossed and do so by feeding the inner webbing loop of one leg strap through the lowest loop of the other one. After the attaching the leg straps, attach the shoulder loops to the spreader bar. Choose the loops that best fit the patients build and make sure there is no creasing.

To create a hammock sling, place the leg straps under both thighs and pass through the black holding straps. One through the outside holding strap and the other one through the inside holding strap. Then attach the leg and shoulder straps to the spreader bar.

A more upright position of the patient is achieved by keeping the leg straps long and the shoulder straps short. The position may be varied by using the different loop adjustments on each of the straps. Always be sure the same position is used for both leg straps and for both shoulder straps.

Unless the individual assessment shows a specific safety reason for not doing so, always ensure that the hoist brakes are off. This reduces any swinging that may occur and allows the hoist to find its own center of gravity for a safe transfer. Also ensure that the sling is not creased or rucked underneath the patient and that the sling is free from all obstacles and all loops are securely attached to the hoist's spreader bar.

Raise just enough so that the patient is only slightly suspended. Then check patient's position, security and comfort before proceeding with the transfer.

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MSP Medical Products B.V. | De Sonman 7, 5066GJ Moergestel, Netherlands